

INSTALLATION INSTRUCTIONS

XL-12

LIC AUDIO 

INSTALLATION INSTRUCTIONS XL-12

- ❑ Place XL-12 on a suitable place – mounted to the wall or standing on the TV-set.
- ❑ Connect signal (TV, radio etc.) to ”INPUT”- connector* on XL-12. When using microphone input, attach microphone close to the speaker of TV radio etc.

INPUTS:

Input 1: 8 mV – 10 V / 25 k Ω (Line)*

Input 2: 8 mV – 10 V / 25 k Ω (Line)*

Input 3: 6 mV – 4,5 V / 7,8 k Ω (Microphone)*

* See opposite side.

- ❑ Connect the appropriate hearing aid (Mini loop XL-22/II or headset to one of the output connectors).

= Headset or mini loop with 2-pin DIN.

= Headset with 3.5 mm tele connector. (NB: Only use 3-pin tele connector, STEREO, on **P**-output).

- ❑ Check that the switch on left side of XL-12 is placed in appropriate setting according to the symbols.
- ❑ Connect XL-12 to the mains.
- ❑ Adjust the volume of the sound source to NORMAL .
- ❑ Check sound by listening.

NB: If necessary the output level can be adjusted by a potentiometer on right side of XL-12. After adjustment wait 30 sec. before listening, due to built in AGC-function.

LIC AUDIO AB


P.O. Box 603, SE-194 26 UPPLANDS VÄSBY

Phn.: + 46 8 590 00 450

Fax.: + 46 8 590 00 490

www.licaudio.se

DRILL PATTERN

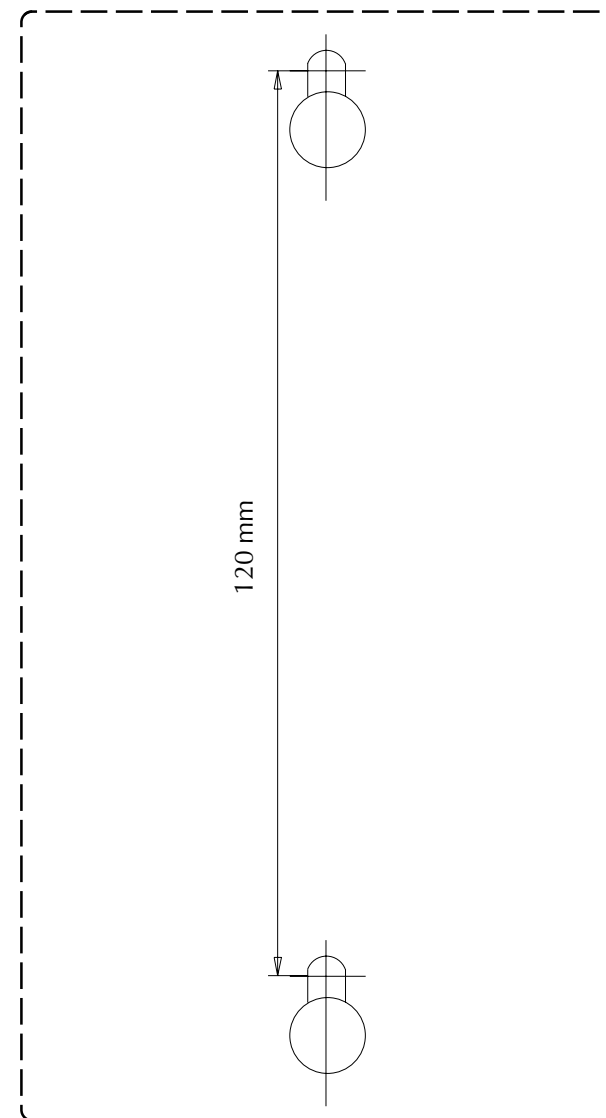

INPUTS

