

INSTALLATION INSTRUCTIONS XL-50 HEADPHONE/LOOP AMPLIFIER

1. Place XL-50 close to the TV-set or radio.
2. Connect loop wire, loop pad or headphone to the output on rear of XL-50.
3. Connect the appropriate input, **MIC** or **LINE**.
LINE input is used with SCART connector or similar.
MIC input is used with the electret MINI microphone or similar. Attache the microphone close to the loudspeaker. Adjust the volume of the TV-set for comfortable listening in the room.'
4. Connect XL-50 to the mains. Check that the LED „POWER” is on.
5. Adjust volume on XL-50 for suitable listening on loop. The LED „LOOP” is on pacing the signal.
6. If necessary adjust „BASS” and „TREBLE”.
7. When an external signal (e.g. door bell) is connected via a relay to „IND”, a tone indication will be present on loop when the external signal is active.